



MADE IN ITALY

# VISIOFOCUS® ● mini ●



Together with Thermofocus and Visiofocus, **VISIOFOCUS•mini•** is the most advanced thermometer a family or a doctor can own to precisely measure body temperature.

Totally hygienic, **without touching the skin**, **VISIOFOCUS•mini•** projects the temperature directly on the forehead and on any other surface.

When you see the temperature in the middle of the two dots, the thermometer is at the right distance. Release the button, keep the thermometer steady while the projection flashes and read the temperature value.

	TOO FAR!	• 36.8°C
	TOO CLOSE!	37.2°C •
	YES: CORRECT DISTANCE!	• 37.0°C •

**PARKER**  
Healthcare

An Australian owned and operated company

# 6 in 1



**Body**

## 6 in 1: **VISIOFOCUS •mini•**

is able to take the body temperature and the temperature from any object and liquid with temperatures between 1 to 80°C (33.8 to 176°F) such as:

**Food**



**Bathwater**



**Room temperature**



**Skin**



**Bottle of wine, etc.**



## TECHNICAL DATA

### VISIOFOCUS•mini• 06700

-NUMBER OF BUTTONS:	3
-ROOM TEMPERATURE DETECTION	YES
-AQCS (Automatic Quick Calibration System)	YES
-MQCS (Manual Quick Calibration System)	YES
-PROJECTION DURATION	10 seconds
-BATTERIES (normally NOT INCLUDED)	2AAA/LR03 alkaline
- BATTERIES' LIFE	3 years or up to 15.000 measurements
-RESOLUTION	0.1
-FOREHEAD MEASURING RANGE	34.0/42.5°C (93.2/108.5°F)
-GENERAL MEASURING RANGE	1.0/80°C (33.8/176°F)
-ACCURACY LEVEL <i>(in instrumental tests according to ASTM E1965-98:2009 standard):</i>	

Celsius		Fahrenheit	
<i>from 34 to 35.9°C</i>	+/-0.3°C	<i>from 93.2 to 96.7°F</i>	+/-0.5°F
<i>from 36 to 39°C</i>	+/-0.2°C	<i>from 96.8 to 102.2°F</i>	+/-0.4°F
<i>from 39.1 to 42.5°C</i>	+/-0.3°C	<i>from 102.3 to 108.5°F</i>	+/-0.5°F
<i>from 1.0 to 19.9°C and from 42.6 to 80.0°C</i>	+/-1.0°C	<i>from 33.8 to 67.9°F and from 108.7 to 176°F</i>	+/-1.8°F
<i>from 20 to 33.9°C</i>	+/-0.3°C	<i>from 68.0 to 93.1°F</i>	+/-0.5°F

-ROOM TEMPERATURE WORKING RANGE 16/40°C (60.8/104°F) accuracy guaranteed

-EXTENDED ROOM TEMPERATURE WORKING RANGE\*

Forehead ("face" button): 10/16°C (50/60.8°F) and 40/45°C (104/113°F)

Other measurements ("home" button): 5/16°C (41/60.8°F) and 40/45°C (104/113°F)

\* *VisioFocus Mini can work also in environments with a temperature between 5 and 15.9°C (41/60.6°F) and 40.1 and 45°C (104.1/113.0°F), but in this case the accuracy is not guaranteed and the message "Lo.5" and the temperature value are shown alternatively.*

Medical Device CE0051

### AVAILABLE PACKAGING



Part number : 06700



[www.parkerhealth.com.au](http://www.parkerhealth.com.au)  
[www.parkerhealth.com.my](http://www.parkerhealth.com.my)  
[www.parkerhealth.co.nz](http://www.parkerhealth.co.nz)



Scan QR code  
to download pdf

PD594